


MULTI-USE CYTOTOXIC DRUG SPILLS KIT

BEFORE YOU START :

Always wear the protective gloves, gown, overshoes, face mask and goggles provided in the kit.

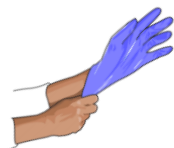

Always improve ventilation in the area of the spill by opening a window and/or door.

Use the disposable forceps to remove any sharps present and transfer to an appropriate sharps container.



LIQUID CYTOTOXIC DRUG SPILLAGES :

Sprinkle CT-Zorb granules over the spill until all moisture is absorbed. Leave for 30 - 60 seconds.



Set up the cytotoxic waste bag close to the spill. Using the Scoop and Scraper collect the spill mixture. Discard all into the cytotoxic waste bag.



Using one of the detergent wipes, clean the affected area after all the spill has been removed.



POWDER CYTOTOXIC DRUG SPILLAGES :

Dampen an absorbent pad using a sterile saline pod and place gently over the spilt material. Dab the surface of the pad to absorb the powder.



Dispose of the used pad into the cytotoxic waste disposal bag.



Using one of the detergent wipes, clean the affected area after all the spill has been removed.



AFTER SPILLAGE REMOVAL :

Place all used materials and protective clothing, gloves last, into the cytotoxic waste bag. Seal bag top using bag tie provided.



NOW WASH YOUR HANDS.

Ensure the bag and all contents are discarded in accordance with your Local Authority Regulations.

Contra Indications of Use:

- ✗ Avoid contact with Hydrofluoric acid
- ✗ Do not use for blood or body fluid spills